

Prepare your NovoPen® 6 or NovoPen Echo® Plus



Remove the cartridge holder

Pull off the pen cap. Twist off the cartridge holder. The piston rod may stick out of your pen. If so, push it in until it stops.



Insert new cartridge

Slide a new insulin cartridge into the cartridge holder with the threaded end first. Twist the cartridge holder back on.



Attach a new needle

Tear off the paper tab. Push the needle straight onto the pen. Turn until it is on tight. Pull off both needle caps. Save the outer needle cap.



Keep a personal injection log If you have a smartphone, tablet, personal computer, blood glucose monitor or another device that supports Near Field Communication (NFC), you can easily transfer your insulin dosing information from your pen to your device. Just make sure to download a compatible app or other software that can read and




NFC location may vary by phone model.

display this information.

Inject



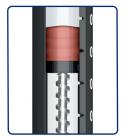

△ ALWAYS test the insulin flow

Select 2 units and press the dose button until the dose counter shows 0. The insulin flow test is complete when you see insulin squirt and there is no longer a gap between the piston rod head and the piston.



Select your dose

Pull out the dose button. Make sure the dose counter shows 0 before you start. Turn the dose button to select the dose you need.



Inject your dose

Insert the needle. Press the dose button until the dose counter shows 0. Count slowly to 6 before you remove the needle from your skin. Safely remove and throw away the needle. Put the pen cap back on.

Check your last dose



Pull out the dose button

Make sure the dose counter shows 0 before you start.



Push the dose button back in

The dose memory will now show the details of your last dose.



Read your dose memory

The example above shows 12 units injected 1 hour,

52 minutes and 3 seconds ago.



NovoPen® 6 and NovoPen Echo® Plus are registered trademarks of Novo Nordisk A/S. © Novo Nordisk Allé, DK-2880 Bagsværd, Denmark. The N Mark is a trademark or registered trademark of NFC Forum, Inc. in the United States and in other countries. HQ22NP600036; Approval date: August 2022

NovoPen® 6) NovoPen Echo® Plus)

NovoPen® 6 or NovoPen Echo® Plus must be prescribed by your healthcare provider. NovoPen® 6 & NovoPen Écho® Plus are intended for patients who have been prescribed a NovoNordisk®

NovoPen® 6 offers a 60-unit maximum dose in 1-unit dose increments. NovoPen Echo® Plus offers a 30-unit maximum dose in 0.5-unit dose increments.

insulin cartridge.