

How to read the dose memory



- More than 12 hours ago (All 12 segments shown)
- 5 units injected



- More than 1 hour ago, but less than 2 (One segment shown)
- 5 units injected



- Less than 1 hour ago (No segments shown)
- 5 units injected



End of battery life

- The NovoPen Echo® memory battery will last four to five years
- When the battery is running low, the message "End" will run across the display when you check your last dose
- Contact your supplier to get a new pen
- You can still use your pen for insulin delivery until you get a new one



Dose memory inactive or battery flat

- Pull out the dose button, then push it back in to activate the dose memory. If the display stays blank, the battery is flat.
- You can still use your pen for insulin delivery until you get a new one



Last injection not registered

- Normally, the dose memory will start working again after your next injection. If not, contact your supplier to get a new pen
- You can still use the pen for insulin delivery without using the dose memory